Lansinoh®

Lansinoh® Einzelpumpset

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EN Lansinoh® Single Pump Set Instructions for Use

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EN / ENGLISH

LANSINOH® SINGLE PUMP SET

Please note these instructions will show you how to use this pump set as a single pump set for Lansinoh® Compact Single Electric Breast Pump as well as using it as part of a double pump set for the Lansinoh® 2in1 Electric Breast Pump or Smart Pump 2.0 Double Electric Breast Pump. Use this pump set only with Lansinoh breast pumps.

- Apart from the tubing, Y connector and L connector, please take everything apart, and boil them for 5 minutes to sanitize. (Lansinoh pumps are a closed system design, therefore, the tubing must not be sanitized.)
- Always make sure all parts are clean and completely dry before assembly.
- Always wash your hands before assembly.
- All parts, apart from the tubing and connectors (Y & L) are dishwasher safe.

Please note: valve and diaphragm are made from silicone and thus need to be treated carefully. We recommend washing these parts by hand in warm soapy water.

| BREASTMILK EXPRESSION SET | ITEM NUMBER |
|--|---------------|
| a Diaphragm Cap | P53410 |
| b Diaphragm | P53402 |
| c ComfortFit® Breast Cushion 25mm / 30.5mm | P53408/P50419 |
| d Breast Cushion Body | P53454 |
| e White Valve | P53401 |
| f 160ml (5oz) Bottle | P53451 |
| g Sealing Cap | P53461 |
| BREAST PUMP PARTS AND ACCESSORIES | ITEM NUMBER |
| h Silicone Tubing Set | P53458 |
| i Pump Connector | P54106 |
| j Y connector | P53405 |
| k L connector | P53453 |

1. ASSEMBLY

(Valid for Lansinoh® Compact Single Electric Breast Pump, Lansinoh® 2in1 Electric Breast Pump and Smart Pump 2.0 Double Electric Breast Pump.)

- Place purple diaphragm (b) on top of breast cushion body (d) as shown in the diagram.
- Then place and twist the diaphragm cap (a) onto breast cushion body (d) enclosing the purple diaphragm.
- Push ComfortFit™ breast cone (c) into breast cushion body (d).
- Gently secure white valve (e) into the bottom of breast cushion body (d).
Note: Use caution when handling and cleaning the small white valve (e).
- Screw collection container (f) onto breast cushion body (d).
- Insert tubing (h) into back of diaphragm cap (a); ensure tight fit.
- Insert tubing connector (i) between the sets. Place tubing into the pump.

2. ASSEMBLING THE TUBING FOR THE LANSINOH COMPACT SINGLE ELECTRIC BREASTPUMP

- Please insert the long tubing (600mm) into the rear of the diaphragm cap.
- Please insert the straight pump connector into the pump motor and connect the other end of the long tube (600mm) to it.

3. ASSEMBLING THE TUBING FOR LANSINOH® 2IN1 ELECTRIC BREAST PUMP AND SMART PUMP 2.0 DOUBLE ELECTRIC BREAST PUMP

You will need two short tubes (300mm), one long tube (600mm), the L-shaped pump connector and the Y-connector

- Please place one short tube (300mm) into the each of the rear of the diaphragm cap.
- Connect both short tubes (300mm) and the long tube (600mm) using the Y-connector

3. Please insert the L-shaped pump connector into the pump motor and connect the other end of the long tube (600mm) to it.

| SYMBOLS | MEANING OF THE SYMBOLS |
|---------|--|
| | Read and follow the instructions for use |
| | Identifies the manufacturer |
| | Indicates the date of manufacture |
| | Indicates the part number of the device |
| | Indicates the batch number |
| | Keep the device away from sunlight |
| | Keep the device dry |
| | Indicates that the packaging contains products intended to come in contact with food. |
| | Indicates to handle the fragile device with care. |
| | Indicates the compliance with the essential requirements of the Council Directive 93/42/EEC of 14 June 1993 concerning medical devices |

This product is a medical device accessory and should be disposed as a medical product according to the local procedures and regulations.

Serious events experienced from using this product should be reported immediately to Lansinoh Laboratories and to the local competent Authority.

Intended Use

Lansinoh® Single Pump Set is intended to express and collect the breastmilk of a nursing woman for the purpose of feeding collected breastmilk to a baby. Lansinoh® Single Pump Set is intended for a single user.

Indications

Lansinoh® Single Pump Set is indicated to elimination of clinical situations such as milk stasis in the breast, engorgement and mastitis caused by insufficient milk discharge, relieve sore and cracked nipples and to bring out flat or inverted nipples. Also, Lansinoh® Single Pump Set enables mothers to provide breast milk for babies who cannot feed directly at the breast for a variety of reasons, such as latch-on problems, premature or unwell babies.

Contraindication

There are no known contraindications.

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Lansinoh® is a registered trademark of Lansinoh Laboratories, Inc. © 2022 Lansinoh Laboratories, Inc.
All rights reserved.
Lansinoh® Laboratories Saglik Gereçleri Tasarim San. Tic. Ltd. Sti.
Lansinoh Laboratories Inc. | lansinoh.de
Lansinoh Laboratories UK Ltd | lansinoh.co.uk
Lansinoh Lab. Inc. Mer. Delaware ABD Türkiye İstanbul Şubesi | lansinoh.com.tr