56mm

Lansinoh.

EN Lansinoh® Hot & Cold Breast Therapy Packs

lansinoh.co.uk

Use hot to help relieve engorgement, plugged ducts, mastitis, and to encourage milk let-down.

Place one pack inside microwave lying flat for time indicated below. Each pack should be heated separately. After testing pack temperature, you may add more time in 5 second increments as needed until desired temperature is reached. Alternative heating instructions can be found in enclosed instructions leaflet. Please read carefully.

Use cold to help relieve pain and swelling due to engorgement. Freeze for at least 4 hours then place into covers before use.

For use with a breast pump: Use hot with any breast pump to encourage let-down and reduce time spent pumping. Heat packs according to heat therapy instructions and wrap around the outside of the breast cushions.

Apply the packs for a maximum of 20 minutes per session. To hold in place, packs can be placed in a bra. Discard pack if punctured. Do not use on infants or allow infants to chew on the packs.

Medical device
Please see instructions for additional important use advices.

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CAUTION: Always test pack temperature prior to application. Prolonged application may cause injury, including burns. Do not use on infants. Do not use if you have poor circulation. Do not use heat therapy on stitches or an open wound. Do not fall asleep using pack. Use only under adult supervision. For extreme, prolonged, or chronic pain, contact a physician. Use only as directed. Heating beyond suggested time may damage or destroy the pack. Do not puncture pack and/or ingest contents. For external use only. Do not place soft covers in microwave.

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Lansinoh.

Supporting breastfeeding mums for more than 35 years

EN TR

Use without pump

Hot: Helps relieve mastitis, plugged ducts, and encourages milk let-down.

Cold: Helps relieve pain and swelling due to engorgement.

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Use with pump

Hot: Use with any electric or manual breast pump to encourage let-down and reduce time spent pumping.

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Lansinoh.

Hot & Cold Breast Therapy Packs

Use Hot: Helps relieve engorgement, mastitis and plugged ducts

Use Cold: Helps relieve pain and swelling due to engorgement

Use with a breast pump to encourage let-down and reduce time spent pumping

Care

2 reusable packs with soft covers

5 060062 997644

TH10400CT0523

Lansinoh Laboratories Inc. Niederlassung Deutschland, Bayerischer Platz 1, 10779 Berlin | lansinoh.de
Lansinoh Lab., Inc. Mer, Delaware Türkiye İstanbul Şubesi | Tel: +90 216 527 0905 | lansinoh.com.tr
Atatürk Mahallesi Ertuğrul Gazi Sk. Metropol, İstanbul A2 Blok Kat: 22 Daire: 340 34758, Ataşehir/İstanbul/Türkiye

Lansinoh® is a registered trademark of Lansinoh Laboratories Inc.

Lansinoh Laboratories UK Ltd,
Ground Floor, West Mayesbrook
House, Lawnswood Business Park,
Leeds, LS16 6QY

Manufactured by:
Health Plus (Shanghai) International Co., Ltd
Room 1-33, Building 6, 4299 Jindu Road,
Minhang District, Shanghai, China

Everbiz GmbH
Landsberger Str.155 80687
München, Germany

NP2 Technology Ltd
8 Saberton Close, Waterbeach,
Cambridge, CB25
9QW, United Kingdom

LOT #

LOT #

Lansinoh.

Supporting breastfeeding mums for more than 35 years

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