

## How to apply C-section Hydrogel Pads

**Step 1**  
Making sure the area is dry & clean from lotions etc

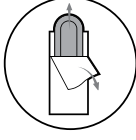**Step 2**  
Open pouch and remove the gel pad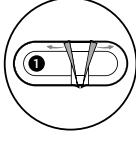

**Step 3**  
Peel back the release paper on the side marked (1)

**Step 4**  
Gently place the gel side of the pad directly over your closed incision site. Apply the pad by massaging the area using a smooth motion from the center outwards

**Step 5**  
Remove release paper on the side marked (2)

## How to remove C-section Hydrogel Pads

**Step 1**  
Use one hand to gently support the padded centre of the pad (light purple)

**Step 2**  
With your other hand, slowly pull the thin film edge of the pad outward, stretching to release the adhesive

**Step 3**  
Continue stretching to release the adhesive while working your way around the outside of the pad

**Step 4**  
Continue until all the adhesive has been released

**Step 5**  
The whole pad will now easily peel away from your skin

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## EN | C-SECTION HYDROGEL PADS

### INSTRUCTIONS FOR USE

**Indications:** The C-section Hydrogel Pads protect the C-section wound minimizing friction from clothing and providing a cooling and soothing effect on application.

**Contraindications:** There are no known contraindications.

**Caution:** Use only as directed. Keep out of reach of children. For external use only. Suitable for use from 10 days after childbirth or when stitches or staples have been removed or have dissolved. If an infection is present or suspected do not use and seek medical advice. Should sensitivity or irritation occur then stop using the C-section Hydrogel Pads and seek medical advice. Wear for a maximum of 72 hours. Single use only.

The C-section Hydrogel Pads may rarely cause pain, skin reaction, infection, foul odour, edema, ulceration, allergic reaction or skin damage on application.

**Directions:**

- Ensure the area is clean and dry before applying the C-section Hydrogel Pad.
- Open pack and remove release paper 1.
- Gently place the gel side of the pad directly over your closed c-section wound.
- Apply the dressing to the area using a smooth motion from the centre outwards
- Remove release paper 2.
- To remove lift gently from the edge, pulling outwards rather than upwards, and in the direction of any hair growth.

- For additional cooling relief, store pads in the fridge before use.
- Do not use the pad with other oil, creams or ointments as it can stop the pad from sticking.

Dispose of pad according to local regulations and guidelines. If the pouch is open before use or has any signs of damage then dispose of pad and do not use. Use the pad as soon as the pouch is opened.

**MD** Serious events experienced from using this product should be reported to the manufacturer and to the local competent Authority

The C-section Hydrogel Pads are intended for single patient use. Re-use outside of the maximum permitted timeframe, or between individuals, may lead to increased risk of infection or cross contamination, and the physical properties of the device may no longer be optimum for intended use.

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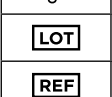
SYMBOLS	MEANING OF THE SYMBOLS
	Single Use
	Do not use if package is damaged
	Consult instructions for use
	Identifies the manufacturer
	Date of manufacture
	Medical device
	DEHP Free
	Latex Free
	Use by date
	Keep away from sunlight
	Keep dry
	Batch code
	Indicates the part number of the device

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Lansinoh Laboratories UK Ltd

Ground Floor, West Mayesbrook House, Lawnswood

Business Park, Leeds, LS16 6QY, UK | Lansinoh.co.uk

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Lansinoh Laboratories Inc.
Saglik Gereçleri Tasarim Sic. Ltd. Sti.,